GENIUS™ 2.0

Receiver-In-Canal Hearing Aid

User Manual

www.miracle-ear.com

Welcome

Thank you for choosing Miracle-Ear hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This manual, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user manual and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Content

Your hearing aids	4
Instrument type	4
Getting to know your hearing aids	4
Components and names	5
Controls	7
Settings	9
Batteries	11
Battery size and handling tips	11
Replacing batteries	12
Daily use	14
Turning on and off	14
Inserting and removing the hearing aid	16
Adjusting the volume	19
Changing the hearing program	20
Special listening situations	21
On the phone	21
Audio induction loops	23
Maintenance and care	24
Hearing aids	24
Exchanging ear pieces	26
Replacing the miniReceiver 2.0	28
Professional maintenance	29

Further information	30
Wireless functionality	30
Accessories	30
Troubleshooting	31
Service information	33

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Your hearing aids

This user manual describes optional features that your hearing aids may or may not have. Ask your Hearing Care Specialist to indicate the features that are valid for your hearing aid.

Hearing aid type

Your hearing aids are the RIC (Receiver-in-Canal) style. The receiver is placed within the ear canal and connected to the hearing aid via a receiver wire. The hearing aids are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.


Getting to know your hearing aids

It is recommended to familiarize yourself with your new hearing aids. With the hearing aids in your hand try using the controls and note their location. This will make it easier for you to feel and press the controls while wearing the hearing aids.

If you have problems pressing the controls of your hearing aids while wearing them, you can ask your Hearing Care Specialist about a remote control option.

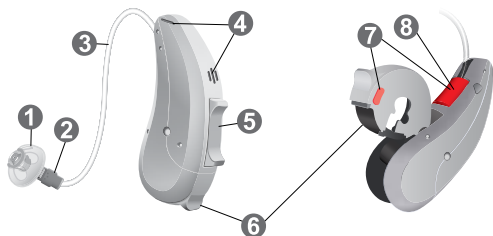Components and names

-
- | | |
|---|--|
| 1 Ear piece | 6 Battery compartment (on/off) |
| 2 Receiver | 7 Side indicator (red = right ear, blue = left ear) |
| 3 Receiver wire | 8 Receiver connection |
| 4 Microphone openings | |
| 5 Rocker switch (standard control), push button or no control (optional) | |
-

You can use standard or custom-made ear pieces.
Your Hearing Care Specialist will indicate your model.

Standard ear pieces		Size
<input type="checkbox"/>		Click Dome™ single (open or closed)
<input type="checkbox"/>		Click Dome semi-open
<input type="checkbox"/>		Click Dome double

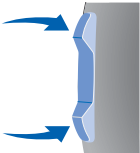
You can easily exchange the standard ear pieces. Read more in section "Maintenance and care."

Custom-made ear pieces	
<input type="checkbox"/>	 Custom shell
<input type="checkbox"/>	 Click Mold™

Controls

With the controls you can, for example, adjust the volume or switch hearing programs. Your hearing aids either have a push button or a rocker switch - or no control at all.

Your Hearing Care Specialist has programmed your desired functions to the controls.

Control	Left	Right
Push button 	<input type="checkbox"/>	<input type="checkbox"/>
Rocker switch 	<input type="checkbox"/>	<input type="checkbox"/>
No control	<input type="checkbox"/>	<input type="checkbox"/>

You can also use a remote control or a smartphone app for remote control.

Push button function mini RIC	L	R
Press <1 second:		
Program change	<input type="checkbox"/>	<input type="checkbox"/>
Press 1-2 seconds:		
Standby/turn on	<input type="checkbox"/>	<input type="checkbox"/>
L = Left, R = Right		

Push button function RIC	L	R
Press <1 second:		
Program change	<input type="checkbox"/>	<input type="checkbox"/>
Volume up	<input type="checkbox"/>	<input type="checkbox"/>
Volume down	<input type="checkbox"/>	<input type="checkbox"/>
Press 1-2 seconds:		
Standby/turn on	<input type="checkbox"/>	<input type="checkbox"/>
L = Left, R = Right		

Rocker switch function	L	R
Press <1 second:		
Program up/down	<input type="checkbox"/>	<input type="checkbox"/>
Volume up/down	<input type="checkbox"/>	<input type="checkbox"/>
Sound balance	<input type="checkbox"/>	<input type="checkbox"/>
Press 1-2 seconds:		
Standby/turn on	<input type="checkbox"/>	<input type="checkbox"/>
Program up/down	<input type="checkbox"/>	<input type="checkbox"/>
Press >3 seconds:		
Standby/turn on	<input type="checkbox"/>	<input type="checkbox"/>

L = Left, R = Right

Settings

Hearing programs
1
2
3
4
5
6

Read more in section "Changing the hearing program."

Features

- Power-on delay** enables whistle-free insertion of hearing aids.
Read more in section "Turning on and off."

- AutoPhone™** automatically switches to the telephone program when the telephone receiver approaches the ear. Read more in section "On the phone."

- The **wireless** functionality enables simultaneous control of both hearing aids. Read more in section "Wireless functionality."

Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. It depends on the battery type how long you have until you have to replace or recharge the battery.

Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.


Battery size: mini RIC 10, RIC 312

- Always use the correct battery size for your hearing aid.
- Remove the batteries if you intend not to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

Replacing batteries


Removing the battery:

- ▶ Open the battery compartment.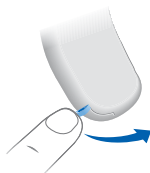

- ▶ Use your finger to remove the battery.

- ▶ Or use a cleaning brush with magnet to pull out the battery. The cleaning brush with magnet is available as an optional accessory from your Hearing Care Specialist.

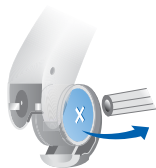Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.

- ▶ Insert the battery with the "+" symbol facing up (refer to the picture).

- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly.

Do not attempt to close the battery compartment by force. It can be damaged.

Daily use

Turning on and off

You have the following options to turn your hearing aids on or off.

Via battery compartment:

- ▶ Turning on: Close the battery compartment.
The **default** volume and hearing program are set.
- ▶ Turning off: Open the battery compartment to the first catch.

After charging:

- ▶ Turning on: Take the hearing aids out of the charger.
The default volume and hearing program remain set.

Via push button or rocker switch:

- ▶ Turning on or off: Press and hold the push button or the rocker switch for several seconds. Refer to section "Controls" for the settings of your controls.
After turning on, the **previously** used volume and hearing program remain set.

Via remote control:

- ▶ Follow the instructions in the remote control's user manual.

After turning on, the **previously** used volume and hearing program remain set.

When wearing the hearing aids, an alert tone can indicate when an hearing aid is being turned on or off.

When the **power-on delay** is activated, the hearing aids turn on after a delay of several seconds. During this time you can insert the hearing aids into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

Inserting and removing the hearing aids

Your hearing aids have been fine-tuned for your right and left ear. Colored markers indicate the side:

- red marker = right ear
- blue marker = left ear


Inserting the hearing aid:

- ▶ Hold the receiver cable at the bend closer to the ear piece.
- ▶ Carefully push the ear piece in the ear canal **1**.
- ▶ Twist it slightly until it sits well.

Open and close your mouth to avoid accumulation of air in the ear canal.

- ▶ Lift the hearing aid and slide it over the top of your ear **2**.

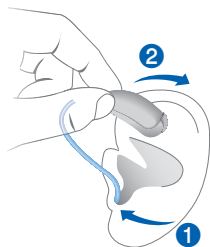CAUTION

Risk of injury!


- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the **right** hearing aid with the **right** hand and the **left** hearing aid with the **left** hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).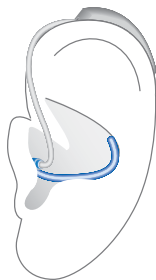

Removing the hearing aid:

- ▶ Lift the hearing aid and slide it over the top of your ear ①.
- ▶ If your hearing aid is equipped with a custom shell or a Click Mold, remove it by pulling the small removal cord toward the back of your head.
- ▶ For all other ear pieces: Grip the receiver in the ear canal with two fingers and pull it carefully out ②.

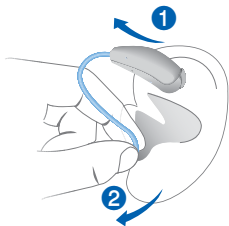Do not pull the receiver cable.

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care."

Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the push button or the rocker switch briefly, or use a remote control.

Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

Refer to section "Controls" for the settings of your controls.

An optional signal can indicate the volume change.

Changing the hearing program

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aid may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the push button or the rocker switch, or use a remote control.

Refer to section "Controls" for the setting of your controls. Refer to section "Settings" for a list of your hearing programs.


If the AutoPhone function is activated, the hearing program switches automatically to a telephone program when the telephone receiver is close to the ear.

Read more in section "On the phone".

Special listening situations

On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.

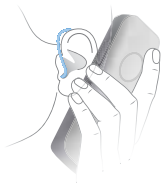Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone or use the automatic program switch.

If a telephone program or the automatic program switch are configured for your hearing aid, it is listed in section "Settings."

Automatic program switch (AutoPhone)

Your hearing aid can **automatically** select the telephone program when you bring the telephone handset close to the hearing aid.

If you want to use this function, ask your Hearing Care Specialist to do the following:

- Activate the AutoPhone function.
- Check the telephone receiver's magnetic field.

For the automatic program switch, your hearing aid detects the telephone receiver's magnetic field.

Not all telephones produce a magnetic field strong enough to activate the AutoPhone function. In this case, you can use the Miracle-Ear Phone Surround magnet, which is available as an accessory, and attach it to the telephone receiver.

NOTICE

Use the approved magnet only. Ask your Hearing Care Specialist for this accessory.

Audio induction loops

Some telephones and public places like theatres offer the audio signal (music and speech) via an audio induction loop. With this system, your hearing aids can directly receive the desired signal - without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.

Ask your Hearing Care Specialist to configure a telecoil program.

- ▶ Switch to the telecoil program whenever you are in a place with an audio induction loop.

Whether the telecoil program is configured for your hearing aid is listed in section "Settings."

Telecoil program and remote control

NOTICE

For hearing aids with wireless connectivity:
When the telecoil program is active, the remote control may cause a pulsing noise.

- ▶ Use the remote control with a distance greater than 4 inches (10 cm).

Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

Hearing aids

Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ Ask your Hearing Care Specialist for recommended drying products.
- ▶ For longer periods of non-use, store your hearing aids with open battery compartment and batteries removed in a drying system to avoid the adverse effects of moisture. A hearing aid dryer is available from your Hearing Care Specialist.

Cleaning

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.

- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

Cleaning instruction for standard and custom made earpieces


Cerumen (ear wax) on the earpiece may affect sound quality. Listen for any changes in how your hearing aid is performing. If wax has built up, it can be easily removed with the cleaning cloth or a soft tissue. Do not use water, alcohol, benzene or any other liquid or household cleaning agent or liquid to clean your earpieces, as it may cause damage.


NOTICE

Never use sharp objects for cleaning, as they may damage the dome. Never use pressure or rotating action while cleaning.

For RIC earmolds, and HP Custom receivers: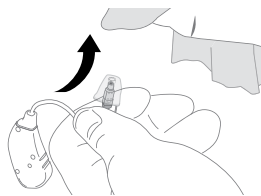


RIC earmold

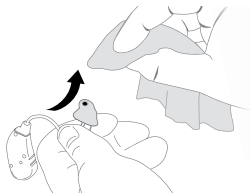HP custom receiver

For Click Domes:

- ▶ Squeeze the tip of the dome.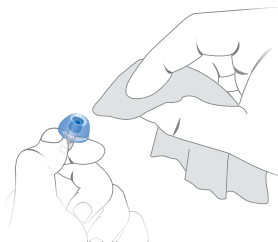


Exchanging ear pieces

Exchange standard ear pieces like your click domes, when showing any signs of wear such as cracking or discomfort approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging depends on the type of ear piece. In section “Components and names,” your Hearing Care Specialist has marked your type of ear piece.


Exchanging Click Domes

If you have problems removing the old Click Dome, use the removal tool or follow the instructions on the packaging of the Click Domes: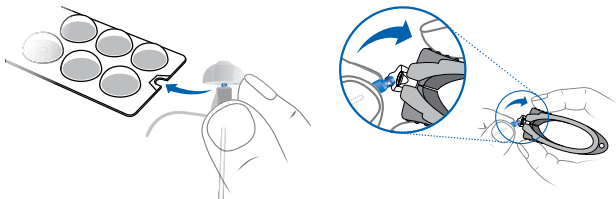

The removal tool is available as an accessory that is available from your Hearing Care Specialist.


- ▶ Take particular care that the new Click Dome clicks noticeably in place.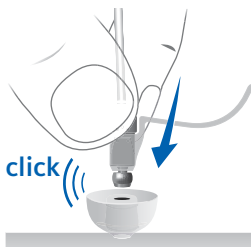


Replacing the miniReceiver 2.0

The miniReceiver 2.0 is easily exchangeable. To replace the miniReceiver, use the exchange tool to open and close the locking bar for reliable securing and no risk of dislodgement.


- ▶ Open the locking bar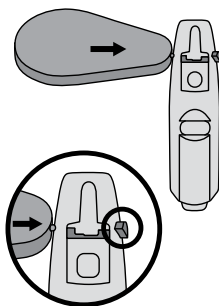

- ▶ Remove/Insert the miniReceiver.

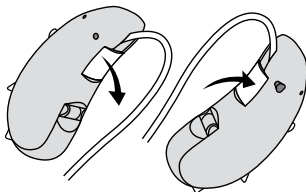- ▶ Close the locking bar.

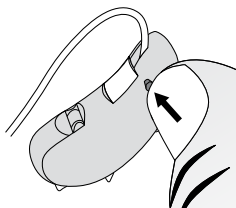Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by a specialist.

Ask you Hearing Care Specialist for your individual recommendation of maintenance intervals and for support.

Further information

Wireless functionality

Your hearing aids may be equipped with wireless functionality. This provides synchronization between your two hearing aids:

- If you change the program or volume on one hearing aid, the other one is automatically changed, too.
- Both hearing aids automatically adapt to the listening situation simultaneously.

Wireless functionality enables the use of a remote control.

Accessories

The type of available accessories depends on the hearing aid type. Examples are: a remote control, an audio streaming device, or a smartphone app for remote control.

Consult your Hearing Care Specialist for further information.

Troubleshooting

Problem and possible solutions

Sound is weak.

- Increase the volume.
 - Replace the empty battery.
 - Clean or exchange the ear piece.
-

Hearing aid emits whistling sounds.

- Reinsert the ear piece until it fits securely.
 - Decrease the volume.
 - Clean or exchange the ear piece.
-

Sound is distorted.

- Decrease the volume.
 - Replace the empty battery.
 - Clean or exchange the ear piece.
-

Hearing aid emits signal tones.

- Replace the empty battery.
-

Problem and possible solutions

Hearing aid does not work.

- Turn the hearing aid on.
 - Gently close the battery compartment completely.
 - Replace the empty battery.
 - Make sure the battery is inserted correctly.
 - Power-on delay is active. Wait several seconds and check again.
-

Consult your Hearing Care Specialist if you encounter further problems.

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

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Derechos de autor ©2018 Miracle-Ear, Inc.
Distributed by Miracle-Ear, Inc.