Audio Clip

User Manual

www.miracle-ear.com

Content

Before you start	5
Introduction	5
Included in package	10
Components	12
Getting started	14
Charging	14
Pairing hearing aids	16
Pairing your smartphone or music player	18
Clear pairings	20

Daily use	21
Turning on/off	21
Adjusting the volume	22
Muting the hearing aid microphones	23
① Making phone calls	24
② Listening to music	28
③ Using as remote microphone	30
④ Using as remote control	33
Important safety information	34
Personal safety	34
Product safety	40

Important information	43
Intended use	43
Status and power indicator overview	44
Explanation of symbols	48
Maintenance and care	50
Technical information	51
Disposal information	53
Conformance information	54

Before you start

CAUTION

Read this user manual thoroughly and completely and follow the safety information in this document to avoid damage or injury.

Introduction

The Audio Clip uses Bluetooth^{®*} technology to provide the following functions.


*The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.

1 Headset for smartphones

Sounds from your smartphone will be streamed to your hearing aids through the Audio Clip.

Your voice is picked up by the Audio Clip microphone and sent to your smartphone.


Read more in section
"Making phone calls".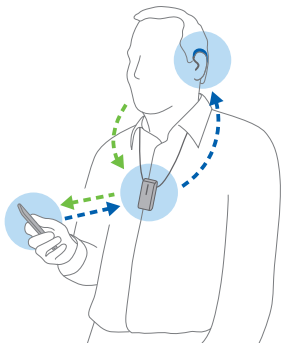

2 Headphones for music

Music from your smartphone or any other Bluetooth® device will be streamed to your hearing aids through the Audio Clip.


Read more in section "Listening to music".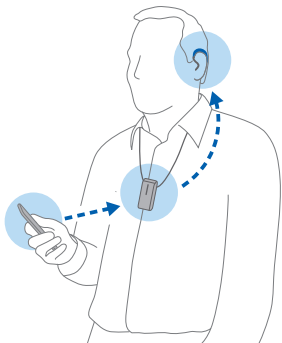

3 Remote microphone

The voice of a chosen speaker will be transmitted wirelessly from the Audio Clip to your hearing aids.

Read more in section
"Using as remote microphone".

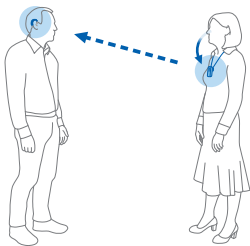

4 Remote control

When the Audio Clip is not streaming sound it can be used as a remote control for your hearing aids.

Read more in section "Using as remote control".

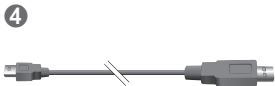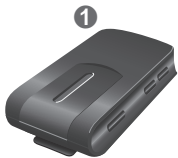
Included in package

- ① Audio Clip
- ② Lanyards (different lengths)
- ③ Pouch
- ④ USB cable for charging
- ⑤ USB power cable and power plug (country-specific)

Components

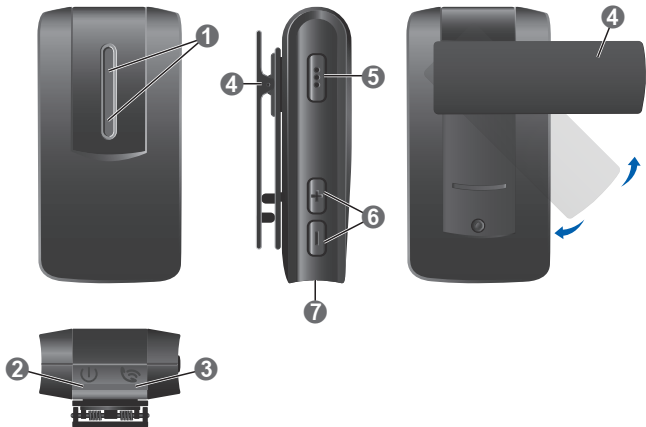- ① Microphones
- ② Power indicator 
- ③ Status indicator 
- ④ Clip (rotating)
- ⑤ Multi-function button 
- ⑥ Volume buttons  and 
- ⑦ USB port for charging

Getting started

Charging

The Audio Clip is partly charged upon delivery. Charge it fully before the first use.

When the power indicator  is red  you have approximately 30 minutes remaining of streaming time.

The power indicator shows the charging status:

 Description

 Battery is low

 Charging (Audio Clip is on)

 Charging (Audio Clip is off)

 Charging is completed (Audio Clip is on)

 Charging is completed (Audio Clip is off)

Further information:

- Typically, it takes 2 hours to fully charge the Audio Clip.
- Make sure that the power plug is easily accessible to remove it from the power supply if necessary.

Pairing hearing aids

Your hearing aids need to be paired once with the Audio Clip. After that, the pairing is saved in the hearing aids and remembered each time they are turned on.

- ▶ Ensure that the Audio Clip is off: Hold down the  button until the power indicator  is off. This takes approximately 6 seconds.
- ▶ Turn the hearing aids off and on again by: Opening and closing the battery doors.

The hearing aids are now ready for pairing. You have 3 minutes to complete the pairing.

- ▶ Turn on the Audio Clip: Hold down the  button until the power indicator  is green . This takes approximately 3 seconds.

The status indicator  flashes yellow .

- ▶ Place both hearing aids close to the Audio Clip (maximum distance 20 cm or 8 inches).

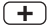
The pairing process starts automatically.

Pairing is finished when the status indicator is yellow .

Pairing your smartphone or music player

Your smartphone (or music player) needs to be paired once with the Audio Clip. After that, the connection is automatically established when the Audio Clip is turned on and within operating distance.

- ▶ Hold down the  and the  button simultaneously until the status indicator  flashes blue . This takes approximately 6 seconds.

The Audio Clip is now ready for pairing. You have 3 minutes to complete the pairing.

- ▶ Set your smartphone or music player to “search for other Bluetooth devices”.
- ▶ On your smartphone or music player, select “Audio Clip” from the list of detected devices.

► If requested, enter the PIN "0000".

Pairing is finished when the status indicator  is blue .

Further information

- The Audio Clip can be paired with up to eight Bluetooth devices. Only one device can be connected at a time.
- When paired with several devices:
 - The Audio Clip will connect to the first device it detects.
 - To switch to another device, deactivate Bluetooth on the device(s) that you do not want to use at the moment.

Clear pairings

To clear all pairings of the Audio Clip with smartphones and music players:

- ▶ Turn off the Audio Clip: Hold down the  button until the power indicator  is off. This takes approximately 6 seconds.
- ▶ Simultaneously hold down the  button and the  button and the  button for 10 seconds.

When the pairings are cleared, the status indicator flashes red  . Your hearing aids remain paired with the Audio Clip.

Daily use

Turning on/off

Turning on:

- ▶ Hold down the  button until the power indicator  is green . This takes approximately 3 seconds.
The status indicator  flashes yellow  until the hearing aids are connected.

Turning off:

- ▶ Hold down the  button until the power indicator  is off. This takes approximately 6 seconds.


Adjusting the volume

Shortly press  or  to adjust the volume.

Depending on your hearing aid type, the volume is adjusted as follows:

- Possibility 1: While sound is being streamed, only the sound from the Audio Clip (phone call, speech or music) is adjusted. While no sound is streamed, the volume of the hearing aids is adjusted.
- Possibility 2: The sound from the Audio Clip (phone call, speech or music) and the volume of the hearing aids are adjusted together.

Muting the hearing aid microphone

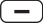Hold down the  button for 3 seconds to mute the hearing aid microphone. Only the sound from the Audio Clip is played in your hearing aids.

Depending on your hearing aid type, you can unmute the hearing aid microphone as follows:

- Possibility 1: Hold down the  button for 3 seconds.
- Possibility 2: Shortly press .

This function may not be available while the Audio Clip is used as a remote microphone.

1 Making phone calls

Prerequisites: the Audio Clip must be connected to a smartphone and it must be worn properly.

Wearing the Audio Clip properly

The Audio Clip should be worn as illustrated in the pictures for best possible performance. It is important that the microphone of the Audio Clip is pointing towards your mouth, and that the Audio Clip is not covered by clothes etc.

Maximum distance:

- Smartphone – Audio Clip: 10 m (33 ft)
- Hearing aids – Audio Clip: 2 m (6.5 ft)

Receiving a call

You will hear a ringtone in your hearing aids. The status indicator  flashes green .

- ▶ To accept/end the call, shortly press  or accept/end the call with your smartphone.

While you are in a call, the status indicator  is green .

- ▶ To reject a call, hold down the  button for approximately 3 seconds.

Making a call

When you call someone with your connected smartphone, the call is automatically streamed to your hearing aids.

While you are on a call, the status indicator  is green .

This function is not available while the Audio Clip is used as a remote microphone.

Further information

- Voice dialing: When connected to a smartphone, double-press  to activate voice dialing (if supported by the smartphone).
- For this kind of connection, the Audio Clip uses the Bluetooth® Hands-Free Profile (HFP).

2 Listening to music

If the Audio Clip is connected to a music player through Bluetooth®, the music will automatically be played in your hearing aids. You do not need to operate anything on the Audio Clip.

While music is streamed, the status indicator  is orange .

Maximum distances:

- Music player – Audio Clip: 10 m (33 ft)
- Hearing aids – Audio Clip: 2 m (6.5 ft)

Starting/stopping music

- ▶ Use your smartphone or music player to start/stop the music.

To stop the music, you can also shortly press  on the Audio Clip.


Further information

- For this kind of connection, the Audio Clip uses the Bluetooth® Advanced Audio Distribution Profile (A2DP).
- If music is streamed from your smartphone, incoming calls are indicated while music is played.

3 Using as remote microphone

The remote microphone function does not start automatically. You have to toggle to the remote microphone mode via the  button.

Wearing the Audio Clip properly

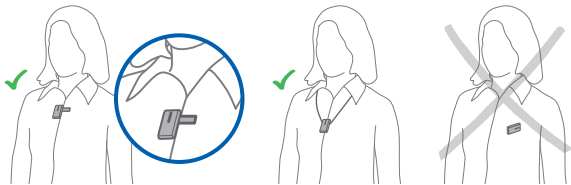The speaker should wear the Audio Clip as illustrated in the pictures for best possible performance. It is important that the microphone is pointing towards the speaker's mouth, and that the Audio Clip is not covered by clothes etc.

Maximum distance:

- Hearing aids – Audio Clip: 20 m (66 ft)

Turning remote microphone on

▶ Press  for approximately 3 seconds.

The status indicator  is magenta . A confirmation tone is played in your hearing aids.

The remote microphones are turned on. Speech from the person wearing the Audio Clip is streamed directly to your hearing aids.

Turning remote microphone off

- ▶ Hold down the  button for approximately 3 seconds.

The status indicator  is blue  (connected to a smartphone) or yellow  (not connected to a smartphone).

Further information

- When using the Audio Clip as a remote microphone there is no connection to the smartphone or to incoming calls.
- The Audio Clip has directional microphones, i.e. microphones that catch the sound from a certain direction.

4 Using as remote control

The Audio Clip can also be used as a remote control for your hearing aids: You can adjust the volume (refer to the section "Adjusting the volume"), mute the hearing aid microphones (refer to the section "Muting the hearing aid microphones") or change to the next hearing program (see below).

Maximum distances:

- Hearing aids – Audio Clip: 2 m (6.5 ft)

Changing to the next hearing program

- ▶ Hold down the  button for approximately 3 seconds.

This function is not available while the Audio Clip is used as a remote microphone.

Important safety information

Personal safety

WARNING

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and cognitively impaired persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

WARNING

Risk of strangulation!

- ▶ Ensure that the lanyard does not get entangled, e. g. in moving mechanical parts of machines.
- ▶ Use only the delivered breakaway lanyard. It will automatically detach when entangled.
- ▶ Do not use the lanyard for children under the age of twelve.

WARNING

Risk of electrical shock!

- ▶ Do not use obviously damaged devices and return them to point of sale.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Specialist for support.

WARNING

The safety of recharging batteries using a USB connector is determined by the external equipment.

- ▶ When the USB connector is connected to equipment plugged into a wall outlet, this equipment should be UL approved or comply with IEC 60065, IEC 60950 or other equivalent safety standards.

WARNING

The safety of the use of this device when connected to external equipment (via auxiliary input cable, via USB cable, or directly), is determined by the external equipment.

- ▶ When connected to external equipment plugged into a wall outlet, this equipment must comply with IEC 60065, IEC 60950 or other equivalent safety standards.

WARNING

Risk of explosion when the rechargeable battery is handled wrong.

- ▶ Do not short circuit, pierce, crush or disassemble the rechargeable battery.
- ▶ Do not place it in oven or fire.
- ▶ Do not immerse it in water.
- ▶ Charge the rechargeable battery between 0 °C and 45 °C.
- ▶ To repair or exchange the rechargeable battery, return the device to your Hearing Care Specialist.

WARNING

Active implants

- ▶ Caution must be taken with active implants. If you have an active implant, contact the manufacturer of your implanted device for information about the risk of disturbance.

WARNING

This device may interfere with measuring and electronic equipment.

- ▶ Do not use your device on airplanes or areas where sensitive or life-supporting electronic equipment is used.

WARNING

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e. g. in mining areas).

Product safety

NOTICE

- ▶ Protect your device from extreme heat.
Do not expose it to direct sunlight.

NOTICE

- ▶ Protect your device from high humidity.

NOTICE

- ▶ Do not dry your device in the microwave oven.

NOTICE

Different types of strong radiation, e. g. during X-ray or MRI head examinations, may damage device.

- ▶ Do not use the device during these or similar procedures.

Weaker radiation, e.g. from radio equipment or airport security, does not damage the device.

In some countries restrictions for the usage of wireless equipment exist.

▶ Refer to local authorities for further information.

NOTICE

Your device is designed to comply with international standards on electromagnetic compatibility but interference with nearby electronic devices could occur. In this case, move away from the source of interference.

Important information

Intended use

This device is intended as a wireless receiver and transmitter of sound between hearing aids and the surrounding environment, including speech and wireless audio devices. This device also enables wearers of wireless hearing aids to control the functionality of their hearing aids.

It is intended to be used by children (> 36 months old) and adults together with their hearing aids.

Status and power indicator overview

Status indicator

Description

Flashing yellow
Searching for hearing aids.

Yellow
Idle mode: Connected to hearing aids.
Not connected to a smartphone or other device.

Flashing blue
The Audio Clip is visible to a smartphone for pairing purpose.

Description

Blue

Phone mode: Connected to a smartphone.

Flashing green

Incoming call.

Green

In a call.

Orange

Streaming music.

Magenta

Remote microphone mode.

Description

Flashing magenta

Reconnecting remote microphone.

Power indicator

Description

Red

Battery is low.

Flashing green

Charging while turned on.

Description

Green fades off

Charging is complete and the Audio Clip is turned on.

Green

Turned on, not charging.

Flashing orange

Charging while turned off.

Orange

Charging is completed and the Audio Clip is turned off.

Explanation of symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Symbols on the device or packaging

CE compliance label, confirms compliance with certain European Directives, refer to section "Conformance information".

Symbols on the device or packaging

EMC and radio communications compliance label Australia, refer to section "Conformance information".

Indicates the legal manufacturer of the device.

Do not dispose of the device with general domestic waste. Read more in section "Disposal information".

Read and follow the instructions in the user manual.

Maintenance and care

NOTICE

- ▶ Do not put your device in water!
- ▶ Do not clean your device with alcohol or benzine.
- ▶ Clean the device as necessary with a soft tissue.
- ▶ Avoid abrasive cleaning liquids with additives such as citric acids, ammonia, etc.

Technical information

Compatible hearing aids

The Audio Clip works with Miracle-Ear specific wireless hearing aids. Your Hearing Care Specialist will advise you on the compatible models.

Operating, transport, and storage conditions

Operating conditions

Temperature	5 to 40 °C (41 to 104 °F)
-------------	---------------------------

Relative humidity	15 to 93 %, non-condensing
-------------------	----------------------------

During extended periods of transport and storage, please observe the following conditions:

Transport and Storage

Temperature -20 to 70 °C (-4 to 158 °F)

WARNING

Risk of injury, fire or explosion during shipment.

- ▶ Ship the Audio Clip according to local regulations.

For other parts, such as hearing aids, other conditions may apply.

Technical data of the power supply

Input voltage:	100-240 V AC
Input current:	max. 0.2 A at 100 V AC
Input frequency:	47-63 Hz
Output voltage:	5 V DC
Output current:	max. 1 A

Disposal information

Recycle devices, accessories and packaging according to local regulations.

Conformance information

The CE mark indicates conformity with the following European directives:

- 93/42/EEC concerning medical devices
- 2014/53/EU RED concerning radio equipment
- 2011/65/EU RoHS concerning the restriction of hazardous substances

Wireless functionality

This device contains a radio transceiver using Bluetooth EDR, Bluetooth low energy (BLE) and a proprietary short range radio technology all working at 2.4 GHz. The 2.4 GHz radio transmitter is weak and always below 17 dBm e.i.r.p. in total radiated power.

This device complies with international standards concerning radio transmitters, electro-magnetic compatibility and human exposure.

FCC and ISED compliance information can be found on the back of the device.

The ACMA compliance mark  indicates conformity with the electromagnetic interference standards set by the Australian Communications and Media Authority (ACMA). Devices with the FCC marking comply with the standards of the FCC regarding electromagnetic interference.

This Class B digital apparatus complies with Canadian ICES-003.

This device complies with Part 15 of the FCC Rules and with ISED's licence-exempt RSSs.

Operation is subject to the following two conditions:

- This device may not cause harmful interference, and
- This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in

a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. There is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

- Connect the equipment to an outlet on a circuit different from that to which the receiver is connected.
- Consult an experienced radio/TV technician for help.

FCC caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

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Derechos de autor ©2020 Miracle-Ear, Inc.
Distributed by Miracle-Ear, Inc.